



How to archive a study

In order to become credentialed using Butterfly, you must save each study that you perform to ensure the QA process can take place.

**** Best Practices: Save your study between each patient
Add patient name and DOB to every study ****


At the END of exam:

- When you have finished capturing your images, tap the **capture reel** in the upper right corner of the imaging screen.



Associate patient details:

- Associate patient details via the worklist by clicking **+Add** and type the first few letters in the patient's last name. Select appropriate patient.

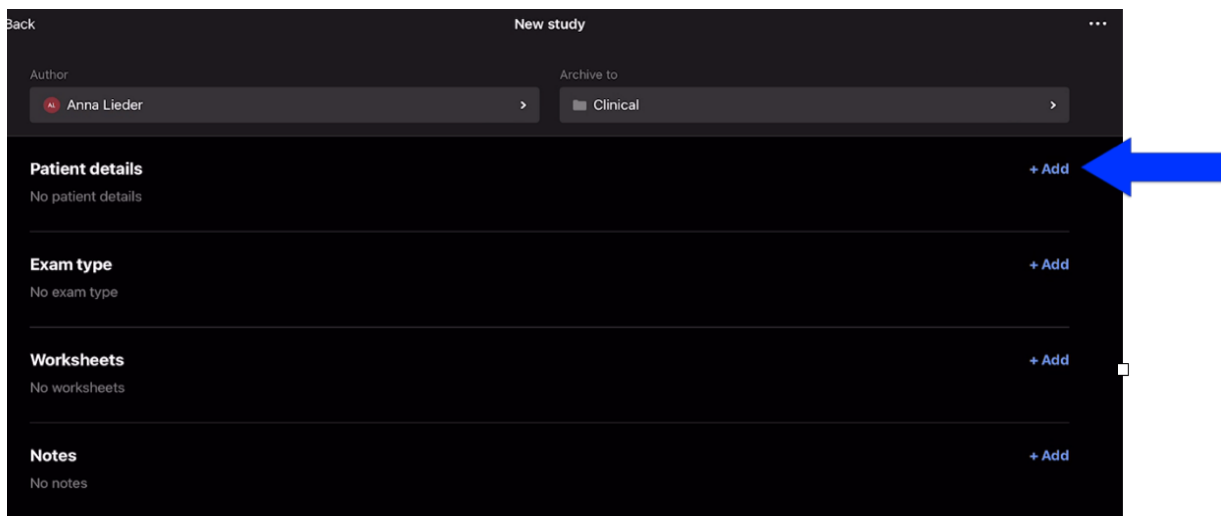

Follow the same steps as above to:

- **+Add** exam type
- **+Add** a worksheet
- Make any necessary notes

To Save study:

- Scroll to the bottom of the Study details page
- Click the blue **Save Study** button

