

## How to Comment on the Butterfly App

While viewing a case on the desktop, the user will see the **Image Actions Menu** along the bottom of the screen. This menu has 5 icons related to different actions.

- Users can click the “Comment” icon to open the comment section and view or add comments.
  - To comment, the user must “tag” the person they want to have a conversation with. To do this, type @username and then the comment. This will allow the system to notify that user via email and on the app’s notification icon that you have tagged them in a comment on a specific image.
- Users can click the “Measure” icon to access the measurement tools which will allow you to add annotations and measurements to images.
- Users can click the “bookmark” button to bookmark a certain image.
- Users can click the “download” button to download a de-identified single image or cine clip.
- Users can select the “delete” button to delete a single image or cine clip.