

R Series[®] ALS

Monitoring

- 1 Attach electrode pads or 3-lead ECG to patient.
- 2 Turn defibrillator dial to MONITOR mode in gray.
- 3 Select LEAD button to chose Lead I, II, III, or pads.




Defibrillation

- 1 Turn defibrillator dial to DEFIB mode in red.
- 2 Use energy select keys to change joules.
- 3 Press CHARGE button.
- 4 When defib charges, clear patient, then press SHOCK ⚡ button.



R Series[®] ALS

Synchronized Cardioversion

- 1 Push Sync On/Off soft key.
- 2 Use ENERGY SELECT keys to select energy.
- 3 Verify Sync and R wave markers.
- 4 Press CHARGE button.
- 5 Press and hold SHOCK  button.



Pacing

- 1 Turn dial to PACER mode in green.
- 2 Dial RATE up or down.
- 3 Dial OUTPUT up or down until capture is achieved.



The R Series Operator's Guide provides information operators need for the safe and effective use and care of the R Series products. It is important that all persons using this device read and understand the information contained in the Operator's Guide.