



News You Can Use

February, 2019

Proper Specimen Collection: Chlamydia/GC/Trichomonas Nucleic Acid Amplification Testing

Testing for *Chlamydia*, *Neisseria gonorrhoeae* and *Trichomonas vaginalis* has transitioned to a new platform.

Please note:

The new Aptima (Multipurpose/Self-Collect and Unisex) collection devices have a screw cap with a foil top. This cap must be **unscrewed** and removed before placing the swab in the tube. Please **do not peel, puncture, or pierce the foil top** or the specimen will be rejected due to concern of contamination.

A detailed chart regarding the Microbiology Swab Conversion and Collection Instructions can be found online via the UR Medicine Labs Test Menu: www.testmenu.com/rochester

Questions regarding proper specimen collection may be directed to the Clinical Microbiology Laboratory at 585-275-7801.

UR Medicine Labs Patient Service Center Locations

The following FF Thompson Patient Service Center locations are now part of UR Medicine Labs:

Patient Service Center Location	Address
Macedon	1033 State Route 31
Mendon	20 Assembly Dr., Suite 104, 14506
Victor	6532 Anthony Dr., Suite B, 14564
Newark	800 W Miller St., Unit 11, 14513
Lima	7325 Community Dr., 14485
Farmington - Thompson Medical Center	1160 Corporate Dr., 14425

Coming Soon: UR Medicine Labs Patient Service Center – Spencerport

How to contact us: email: labservicesoutreach@urmc.rochester.edu

Client Services Call Center: (585) 758-0510, option 3

Go to: www.urmedicinelabs.org

Test Menu: www.testmenu.com/rochester